


# PolyMem<sup>®</sup>

## Finger/Toe Dressings

Help improve recovery  
after injury!

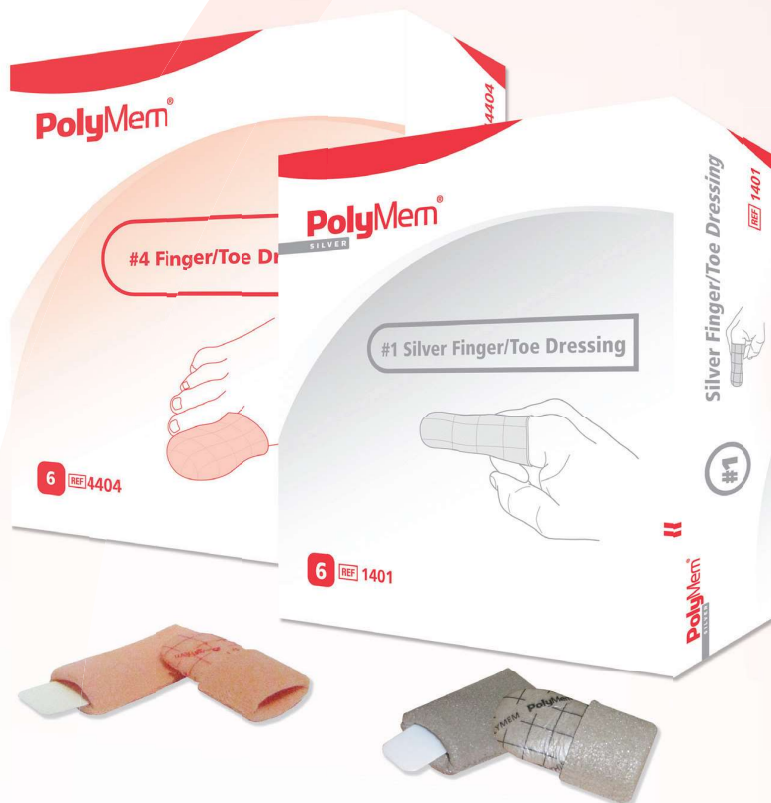

# Made to roll comfortably onto a finger or toe right out of the package.


PolyMem Finger/Toe dressings help reduce edema, bruising, pain and inflammation when applied to open or closed injuries. When applied to open wounds the dressings continuously cleanse, fill, absorb and moisten wounds. The dressings help relieve both persistent and procedure related pain associated with the injury throughout the healing process.

## THE IDEAL CHOICE FOR MANAGING:


- Sprains
- Strains
- Contusions
- Abrasions
- Lacerations
- Burns
- Ulcers
- Matricectomies




1 Measure to determine length of dressing needed, cut off excess.

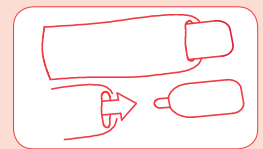

2 Remove the insert from the rolled end and discard.

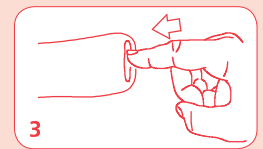

3 Insert the finger into the rolled end of the dressing.

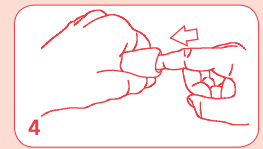

4 Push the finger into the dressing and begin rolling.



5 Roll the dressing on the finger.



6 The dressing should fit securely on finger or toe.

This is an overview. Please see package insert for complete instructions.

Ref	Description	Dimensions	Packaging
4401	#1 Finger/Toe Dressing	#1 small; circumference 1.8" - 2.2" (46.7mm - 57.0mm)	6 Per Box 5 Boxes Per Case
4402	#2 Finger/Toe Dressing	#2 medium; circumference 2.2" - 2.6" (57.0mm - 67.2mm)	6 Per Box 5 Boxes Per Case
4403	#3 Finger/Toe Dressing	#3 large; circumference 2.6" - 3.0" (67.2mm - 77.4mm)	6 Per Box 5 Boxes Per Case
4404	#4 Finger/Toe Dressing	#4 extra large; circumference 3.0" - 3.4" (77.4mm - 87.6mm)	6 Per Box 5 Boxes Per Case
4405	#5 Finger/Toe Dressing	#5 XXL; circumference 3.4" - 3.8" (87.6mm - 97.8mm)	6 Per Box 5 Boxes Per Case
1401	#1 Silver Finger/Toe Dressing	#1 small; circumference 1.8" - 2.2" (46.7mm - 57.0mm)	6 Per Box 5 Boxes Per Case
1402	#2 Silver Finger/Toe Dressing	#2 medium; circumference 2.2" - 2.6" (57.0mm - 67.2mm)	6 Per Box 5 Boxes Per Case
1403	#3 Silver Finger/Toe Dressing	#3 large; circumference 2.6" - 3.0" (67.2mm - 77.4mm)	6 Per Box 5 Boxes Per Case
1404	#4 Silver Finger/Toe Dressing	#4 extra large; circumference 3.0" - 3.4" (77.4mm - 87.6mm)	6 Per Box 5 Boxes Per Case
1405	#5 Silver Finger/Toe Dressing	#5 XXL; circumference 3.4" - 3.8" (87.6mm - 97.8mm)	6 Per Box 5 Boxes Per Case

PolyMem wound care dressings are not made with natural rubber latex



Ferris Mfg. Corp. 5133 Northeast Parkway, Fort Worth, TX 76106-1822 USA  
 Toll Free USA: 800-POLYMEM (765-9636) • International: +1 817-900-1301 • www.polymem.com  
 Trademarks are owned by or licensed to Ferris. The marks may be registered or pending in several countries.  
 © 2014-2018 Ferris Mfg. Corp. MKL-557 R6 0218

MADE IN USA

